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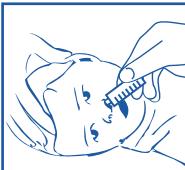
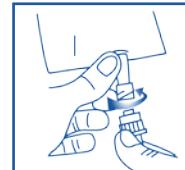
NASIR BABY is a CE medical device specifically designed for hygiene of the nasal cavities of children up to the age of 3 years. Nasir Baby is intended to facilitate cleansing in cases of nasal congestion caused by a cold, sinusitis, rhinitis, respiratory allergies and to help the gradual removal of nasal secretions. The use of NASIR BABY is an effective method for removing excess mucus and preventing secondary infections and complications.

## DOSAGE

-Breast feeding children of 6-36 months. Apply to each nostril 1-2 times a day (2-3ml)  
Children of 1-3 years. Apply to each nostril 2-3 times a day (5-10ml)  
In cases of very severe congestion, we recommend contacting your doctor before starting treatment

## INSTRUCTIONS FOR USE

1. Carefully remove the first protective pouch
2. Warm the bag with the solution to a temperature of approx. 35 - 37 °C by placing it in warm water. Unscrew the protective cap and break the breakable cone.
3. Use a syringe to suck the solution from the bag.
4. Connect the NASIR BABY device to the syringe making sure that it locks home.
5. Press the plunger and make sure that a small quantity of solution flows through the NASIR BABY.
6. Position the anatomical nozzle well into the nostril and inject the solution into the nasal cavity making sure that the child's mouth is always open during the treatment until the solution exits from the other side. In breast-feeding children, given the difficulty of keeping the mouth open, we recommend keeping the head tilted to one side and lower than the shoulders.
7. We recommend blowing the nose after nasal lavage.

## WARNINGS

- If small amounts of the solution reach the throat and cause the child to cough, stop the treatment and have the child spit out or swallow any excess. In small doses, the solution is not toxic.
- If the cough becomes worse, we recommend a few gentle slaps on the child's back (between the shoulder blades).
- If the child reports symptoms of discomfort in the ears or feels a blockage, stop the treatment.
- In children, due to difficulty keeping the mouth open, we recommend to keep their head forward and lower than the shoulders.
- Do not use the product if the child is suffering from ongoing or

recently stopped nose bleeding.

- Do not use in cases of specific hypersensitivity to any component (of the lavage solution).
- Do not use after the expiry date (of the lavage solution).
- Do not use if the device is damaged.
- This product is for external use only.
- The solution can be used within 48 hours of opening if it is correctly stored in a dry place away from light and sources of heat.

## LIMITATIONS

- The sterility and expiry date shown on the solution refer to unopened product that has been correctly stored.
- Store at between + 5 °C + 35 °C away from direct sunlight and sources of heat
- This device is intended for one patient only.
- After treatment, rinse the NASIR BABY device inside and out with normal drinking water and store it in a dry place away from sunlight and sources of heat.
- The next time you want to use NASIR BABY, first disinfect by completely submerging the device in AMUCHINA® solution at 2% (20 ml of product in one litre of water). Let the disinfectant act for 15 minutes. Rinse the device thoroughly inside and out with drinking water. By taking these precautions, you can use the NASIR BABY device continuously for a maximum of 3 months after opening.
- Keep out of the reach of children
- Use this product for its intended use only.
- After use, dispose of the device as normal urban waste

## SYMBOLS AND CE MARKING ON THE LABEL MISSING

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